



## Drug Test Kit Instructions



1.



















Have the donor provide a specimen and return the cup to the collector. Check the temperature strip to assure temperature is between 90°F-100°F.

2.

Replace the cap and set the cup on a flat surface and remove label.


3.

Place the adulterant color guide next to the adulterant strip on the cup. If any of the test pads match any of the colors in the "abnormal" column, adulteration is indicated and the test is invalid.

TEST AND READING TIME	ABNORMAL		NORMAL
	LOW	HIGH	
CREATININE 45 SECONDS	 Neg  10		 20  50  100  200mg/dl
SPECIFIC GRAVITY 45 SECONDS	 1.000	 $\geq 1.030$	 1.005  1.015  1.025
pH IMMEDIATE	 2  3	 $\geq 10$	 4  5  6  9


4.

Interpret results. Negative results can be interpreted as soon as all test lines are visible. Do not interpret positive between 5 and 7 minutes.

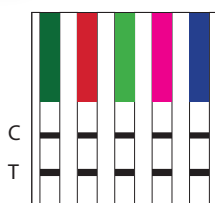


### NEGATIVE

A negative result is indicated by the presence of both a "C" control line and a "T" test line for each designated drug.

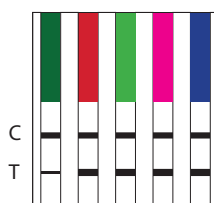


### NEGATIVE

The presence of a "C" control line and a very light "T" test line indicates a negative result. Any indication of a "T" test line is interpreted as a negative result.

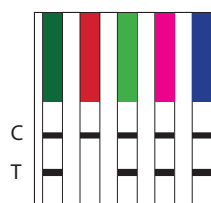

### POSITIVE

A positive result is indicated by the presence of a "C" control line and the absence of a "T" test line. Wait 5 minutes to read positive results.

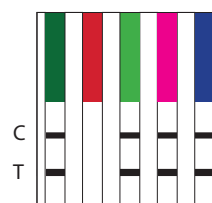

### INVALID

An invalid result is indicated when the "C" line is completely missing from one or more test windows. If this happens, run another test.



## Drugs Commonly Tested For

DRUGS TESTED FOR		COMMON STREET NAMES	MINIMUM DETECTION TIME	MAXIMUM DETECTION TIME
THC	MARIJUANA	POT, WEED, GRASS, REEFER, MARY JANE, GANJA	2 HOURS	UP TO 5+ DAYS
COC	COCAINE	COKE, BLOW, ROCK, CRACK, SNOW, YAYO	1-4 HOURS	2-4 DAYS
MOP	MORPHINE	OPIATES, HEROIN, H, DOPE, BLACK TAR, SMACK, VICODINE, PAIN PILLS, NORCO, VIKES	2 HOURS	2-3 DAYS
AMP	AMPHETAMINE	ADDERALL, RITALIN, VITAMIN R, R-BALL, SKIPPY, KIDDIE COKE, SMARTIES	2-7 HOURS	2-4 DAYS
MET	METHAMPHETAMINE	CRYSTAL METH, CRISTY, TINA, CRANK, GLASS, ICE	2-7 HOURS	2-4 DAYS
PCP	PHENCYCLIDINE	ANGEL DUST, OZONE, ROCKET FUEL, LOVE BOAT, HOG, WACK	4-6 HOURS	7-14 DAYS
BZO	BENZODIAZEPINES	XANAX, KLONOPIN, VALIUM, ATIVAN, ZANNIES, BRICKS, BENZOS, BARS	2-7 HOURS	1-4 DAYS
BAR	BARBITURATES	BARBS, DOWNSERS, SLEEPERS, TOOTSIES	2-4 HOURS	1-3 WEEKS
BUP	BUPRENORPHINE	SUBOXONE, SUBTEX, METHADONE, BUPRENEX	4 HOURS	1-3 DAYS
MDMA	ECSTASY	X, MOLLY, MOON ROCKS, ROLLS, SCOOBY SNACKS, CANDY	2-7 HOURS	2-4 DAYS
MTD	METHADONE	METHADOSE, DOLLIES, DONE, METH, PHY, JUNK, FIZZIES	3-8 HOURS	1-3 DAYS
OXY	OXYCODONE	OXY, OC, OX, 512'S, HILLBILLY HEROIN	1-3 HOURS	1-2 DAYS