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Pregnancy Test Operating instructions


This product is composed of the glass fiber strips of monoclonal antibody against human chorionic gonadotropin (hCG), anti-mouse IgG solid cellulose nitrate membrane and the borders of absorptive colloidal gold - monoclonal antibody against hCG. It adopts the principles of double antibody sandwich method and immunochromatography to test the hCG in the urine.

Intended Use : used for early diagnosis of pregnancy

Model: Midstream II

User: for lay person

Contents: each pouch contains one test stick and one desiccant
Each box contains one pouch and one operating instruction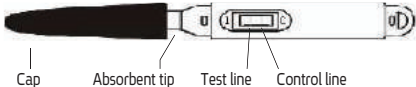

Test Procedure

Before the use please prepare a timer.

1. Remove the midstream from the foil pouch.
2. Remove the cap and place on the other end to expose the absorbent tip.
3. Hold the stick by the thumb grip with the exposed absorbent Cap Absorbent tip Test line Control line tip pointing downward. Urinate on the absorbent tip only until it is thoroughly wet.
4. Lay the test stick down on a flat surface with the windows while you wait for the test result.
5. Read the result within five minutes.
6. Discard the test device after single use in a dustbin.


Interpretation of results

Negative: If only one pink line appears in the control area, you can assume that you are not pregnant.

Positive: If two pink lines appear both in the control area and test area, you can assume that you are pregnant.

Inconclusive: If there is no distinct pink-purple colored band visible both in the TEST area and CONTROL area the test is inconclusive. It is recommended that in this case the test should be repeated.

If the problem persists, contact your local distributor.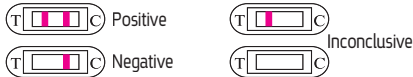

Warnings & Precautions

For in vitro diagnostic use only

1. As with all diagnostic tests, a definitive diagnostic should not be based on the results of a single test, but should only be made by the physician after all clinical and lab findings have been evaluated.
2. Read directions for use carefully before performing this test. Pay attention to the position of the C and T line .
3. Do not use beyond the labeled expiry date.
4. Do not reuse the test devices. Discard it in the dustbin after single use.
5. Do not use if pouch is damaged or opened.
6. Do not touch the membrane located within the windows.
7. Once open the pouch, the test device should be used immediately. Prolonged exposure to ambient humidity will cause product deterioration.
8. Treat urine samples and used devices as if they are potentially infectious. Avoid contact with skin.