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User Guide

Transportador de Oócitos TO-12 Compact - WTA

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Index

WARNING	15
Copyright	15
PRESENTATION	15
INCLUDED ITEMS	16
PRODUCT APPLICATION	17
TECHNICAL FEATURES	17
FUNCTIONS' SPECIFICATION	18
IMPORTANT PRECAUTIONS	19
Repairs	19
Cares and Transport	19
Cleaning	19
Objects Falling Inside the Equipment	19
Power Cable.....	19
Rain and thunderstorms.....	19
Fuses.....	19
Battery.....	19
INSTALATION, SETTING AND OPERATION	20
Temperature Select	20
Maximum and Minimum Temperature During the Transport	20
Audible Temperature Alarm	20
Battery Charge	20
Energy Economy Mode (disables display and alarm sound)	21
Battery Level Warning	21
LIMITED WARRANTY	22
PROBLEM SOLUTION	24
TECHNICAL ASSISTANCE SERVICE/CUSTOMER SERVICE	24

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[wtafiv.financieiro](https://www.wtafiv.financieiro)

WARNING

To prevent accidents and/or damage, read and follow carefully this instruction guide before using the product.

Warning for dangerous voltage presence.

Warning for important operating instructions.

Copyright

Oocytes Transporter TO-12 Compact User Guide

WTA, Cravinhos, State of São Paulo, Brazil.

The content of this guide is protected by copyright laws and cannot be reproduced, distributed, disclosed neither stored without prior written permission of WTA - Watanabe Tecnologia Aplicada Eireli - EPP.

In order to obtain authorization, additional information or to clarify doubts, please contact our Technical Assistance Department by e-mail, letter or phone.

*Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

Phone: +1 (979) 676 - 1968

e-mail: sales@wtavet.com.br

©WTA - Watanabe Tecnologia Aplicada Eireli - EPP - All rights reserved

Comments

The information, illustrations and specifications contained in this user guide assume the latest features of the product available at the time of its publication.

Due to the use of new technologies and raw materials, WTA reserves the right to change the design and/or technical specifications of the product always as necessary and without prior notice, without incurring in any obligations.

This manual is an integral part of the product. Therefore, the consumer should save it for any queries.

PRESENTATION

Congratulations on your choice. You have just purchased a WTA product, a brand that adds quality and technology in what it manufactures.


WTA is a Brazilian company specialized in solutions for animal reproduction. Our products are developed for working with cattle, sheep and horses, with recognized approval by the most renowned specialists in the world.

Its anodized aluminum frame offers, besides an elegant design, conditions that facilitate its internal and external cleaning.

For the best performance of the product we recommend reading this manual.

ACCESSORIES


When acquiring your equipment, be sure that these accessories are intact and included in the package:

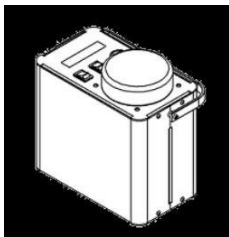Oocytes Transporter
TO-12 Compact


Vehicular cable

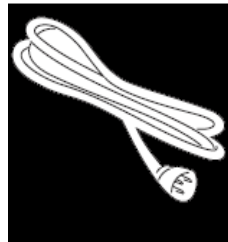
Power cable

Power supply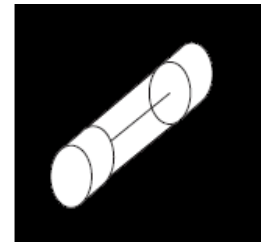

2 Fuses 15A
30mm

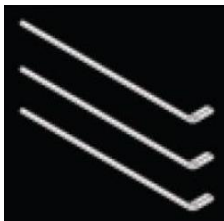12 Handles

Note: If any accessory is missing or defective, please contact the WTA Technical Assistance, within 3 days counted from the product delivery date.

PRODUCT APPLICATION

The Oocytes Transporter TO-12 Compact is an equipment that offers safety to transport oocytes from the field to the IVF laboratory.

temperature during transport, as well as an internal battery that guarantees autonomy of 24hs * of continuous operation, with temperature variation of $\pm 1.0^{\circ}$ F. Suitable for carry-on luggage allows you to carry 24 WTA microtubes of 1.5 mL ($\varnothing = 14.2$ mm).

* Tested at 96, 8°F without interruption at an environmental temperature of 77°F, full load and first heating block connected to power grid

TECHNICAL FEATURES

- Equipment manufactured in aluminum.
- Dimensions (W x L x H) 212 x 125 x 200mm.
- Approximate gross weight: 11.57 lb.
- Maximum power: 64,5W.
- AC Power: 110~240 Vac - 50/60Hz -1.4A.
- DC equipment supply: 15 Vdc; 4.3 A. or 12Vdc (vehicular)
- Temperature adjustment from: 93.2°F to 102.2°F.
- Temperature tolerance of $\pm 1.0^{\circ}$ F.
- Li-Ion Battery: 11.25Vdc; 8850mAh; 99.6Wh.
- Alarm of temperature indicator.
- Cavities number: 12 ($\varnothing = 14.5$ mm).
- Battery charge indicator.
- Product warranty: 2 years.

FUNCTIONS' SPECIFICATION

01 - Display

02 - Switch On/Off

03 - Handle

04 / 05 - Input DC or vehicular cable

06 - Handle

07 - Lid

08 - Menu





09 - Switch + / -

10 - Handles

11 – Storage block

12 - AC Power supply

13 - Output DC

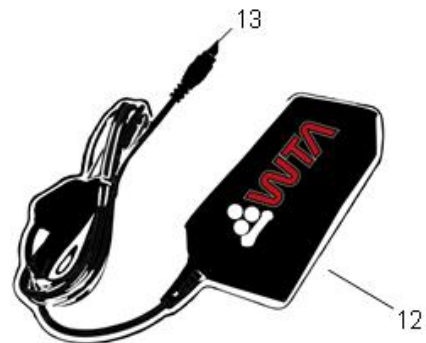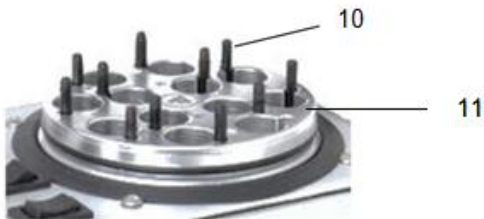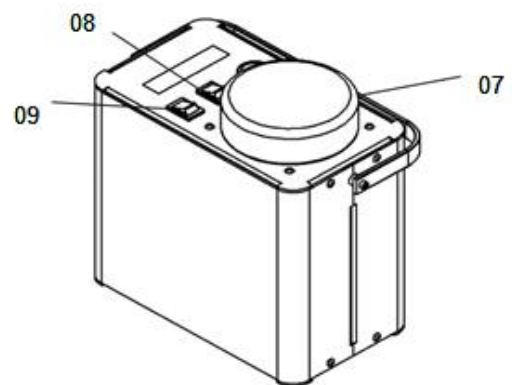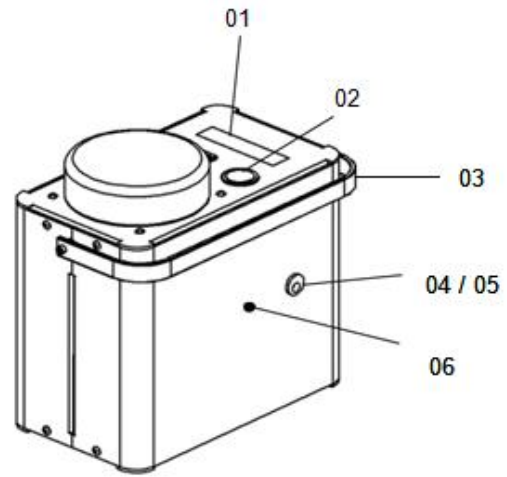IMPORTANT PRECAUTIONS

Repairs:

Do not open the device. All repairs and maintenance should be performed by authorized professionals or by WTA service.

Cares and Transport:

Avoid shaking during transport which can damage it. Make sure that the equipment is properly seated and protected against any accidents.

Cleaning:

Clean the exterior with a soft cloth slightly moistened with water. Do not use alcohol, thinner or other solvents as they will cause damage device's finishing.

Objects Falling Inside the Equipment:

Do not place recipients or cups containing liquid, or coins, paper clips or other metallic objects on the control module. If any liquid or any objects falls inside the equipment, turn off the power immediately and contact the service center.


Power Cable:

Do not step on the power cable or put any kind of objects (furnishing, etc.) over it.

Rain and thunderstorms:

In storms with electric discharge events disconnect the equipment from the AC power.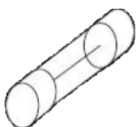

Fuses:

To change the fuse, loosen the metal part of the connector to the cigarette lighter and replace the damaged fuse.

Battery:

In order to make the exchange, it is recommended that this service be done exclusively by authorized technical assistance.

INSTALATION, SETTING AND OPERATION

Temperature Select

Whenever the equipment is turned on, after booting, the display (01) will indicate the last setting temperature. In case that is necessary to modify this value, press "+" or "-" (09) button to increase or decrease the temperature. After selecting temperature, press the "MENU" (08) button to confirm. In case that any button is not pressed for an interval of 10 seconds, the equipment will initialize the process of temperature and heating block control with the last temperature adjusted.

Note: After turning off the equipment, wait 5 seconds to turn it on again.

Maximum and Minimum Temperature During the Transport (T. max and T. min)

With the equipment in operation, press and hold it for 5 seconds the "MENU" (08) button. The display (01) will show T. max and T. min with their values, where T. max is the highest temperature hit and T. min the smallest one. T. Max and T. Min values are saved in memory only after the equipment achieves the temperature set. For reset these values, simply turn off and turn on the equipment again.

Audible Temperature Alarm

The audible alarm will sound when there is a $\pm 2,0^{\circ}$ F temperature variation.

Battery Charge

To achieve full charge, insert the connector (13) into the equipment (04 - 05) and then connect the charger (12) to the mains (110 ~ 240Vac) using the power cable. It is also possible to charge the battery using the car cable (12Vdc). To fully charge the battery, keep the equipment connected to the mains for at least 8 hours, preferably with the equipment switched off. To follow the charging, check the "Indicator Led" (06):

- red = loading
- green = full charge.

Energy Economy Mode (disables display and alarm sound)

With this function is possible to transport with a higher energy saving because disables the display and the alarm sound, so there is a lower battery consumption.

To access this function, by the work screen, keep pressed the "+" (09) button and then hold the "MENU" button (08), the display will disable the display and the audible alarm.

To exit from the economic energy mode, hold the button "MENU" (08), and then hold the "-" button (09) for 10 seconds, so the display (01) and the audible alarm will run again.

Note: To maintain the equipment operating in this mode, hold the "ON / OFF" button at "ON" (02).

If the unit is turned off (key "ON / OFF" on "OFF" position) its position is memorized in economical way, therefore, when it is turned on again (ON / OFF key in "ON" position) the equipment is automatically initiated with the function economic mode activated (display and alarm sound off), with the last working temperature adjusted.

Battery Level Warning

When the battery charge reaches 0% the equipment will be issuing beeps to indicate to the user that the machine is under a load of low battery, this warning will sound around 5 minutes. After this time, the battery charge level is too low, then the equipment is turned off to avoid a large wear on battery.

Note: Never leave your battery reach the critical level of 0%, because if this happens, it will suffer a high wear, thus reducing its life cycle.

WARRANTY

WTA - Watanabe Applied Technology Eireli - EPP follows strict internal quality control standards in the products manufacture. In addition to the legal guarantee of 90 (ninety) days, provided for in the Consumer Protection Code, it guarantees its customers an additional 21-month Warranty, counted from the date of issue of the purchase invoice, provided that the equipment has been installed and according to the guidelines contained in the User's Manual.

Even if ownership of the equipment is transferred, the warranty will continue to be valid according to the terms and conditions specified above.

This product has been carefully evaluated at every stage of the manufacturing process. This warranty will always accompany you, valid only in the country where the product was purchased.

It is understood by Additional Warranty, the free repair of the product and / or the replacement of parts that, according to technical opinion of the WTA, present a manufacturing defect.

Only WTA Technical Assistance professionals are entitled to repair defects for the guarantee, by presenting the sales tax (original) to the first buyer / consumer.

The Warranty Service

- Assistance in WTA technical assistance.
- Parts used for maintenance, subject to the exceptions provided in the item "Exclusion of Warranty".

The product should be sent directly to the WTA observing that:

- The product must be properly packed, if possible in its original packaging, together with the manual, cables and other accessories that have been delivered at the time of purchase.
- The product will be pre-evaluated by WTA technicians prior to repair.
- Shipping, freight and packaging costs will only be charged to WTA during the legal guarantee period (90 days). After this period such costs will be the customer's responsibility.

Voiding Guarantee

This warranty does not cover defects or defects arising from the following situations:

- Installation and use of the product in an improper way or in disagreement with the respective guidelines and information described in this Manual, including installation, external adjustment, cleaning and use of accessories not in accordance with technical specifications.
- Defects caused by accidents, falls, atmospheric phenomena (rains, floods, electric discharges), excessive humidity (water, liquids in general), salinity, sinister (theft or theft), loss or improper transport of the product.
- Installation, adjustment, repair or servicing by persons outside WTA service.
- Invoice and / or serial number adulterated or withdrawn.
- Modifications, alterations or violation of the original product.
- Product connected to the mains improper or subject to excessive fluctuations.
- The product is purchased in balance, showcase or demonstration, this being characterized by means of identification in the invoice of purchase and / or price lower than the one practiced in the market.

Important: WTA is not liable for damage to equipment during transportation. When purchasing your product make sure on the delivery act that it is not defective or with apparent defects. If you notice any irregularities, return the product immediately.

This Warranty Statement Does Not Include

- Periodic reviews.
- Displacement and expense of technician by authorized WTA service.

Responsibility

The WTA has no control or influence over the conditions under which the equipment is used, as well as the methodologies employed. Thus, WTA assumes no responsibility for the results, use and / or performance of the equipment (including: damages for loss of profits, interruption of business activity, loss of business information or other pecuniary losses). WTA expects that the equipment will be used by trained personnel. The purchaser acknowledges that the equipment is subject to all terms and conditions described in this manual.

PROBLEM SOLUTION

Often what at first appears to be defective may be something simple to be solved. If you notice any problem related to the equipment operation, check the procedures listed below.

Malfunctioning on the Power Grid

- Make sure the power cable is properly connected.
- Disconnect it from the power, then wait 5 seconds and connect it again.

Equipment Do not League

- Check that the equipment is switched on, ie the power button must be pressed.
- Verify that the equipment is in the Low Power mode. If so, disable the function.
- Check the battery charge level, it may be low.

Device Does not Work on the Vehicle

- Make sure that the cable is properly connected.
- Make sure that the fuse cord is not broken

If the problem persists, please contact our Technical Assistance Service by making a detailed description of the problem.

TECHNICAL ASSISTANCE SERVICE/CUSTOMER SERVICE

If there is any doubt or if it is necessary additional information about warranty, assembly, installation, adjustment or utilization of the product, please contact our Customer Service Department:

Opening hours: From Monday to Friday, 8:30 am to 5:30 pm (GMT -03:00)

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Images contained in the Guide are merely illustrative.

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Todos os direitos reservados | All rights reserved