**Instructions for Use**

The micropipettes are tools used in Assisted Reproductive Technology (ART) laboratories on micromanipulation procedures that have been developed to aid infertile couples achieving a pregnancy.

**Indications for Use:**

The Intracytoplasmic Sperm Injection (ICSI) Injection Pipettes and Spermatid Injection Pipettes are used to inject a single sperm or spermatid (immature sperm cells) into an oocyte.

The Holding Pipettes are used to hold an oocyte or embryo in position during micromanipulation procedures.

The Partial Zona Dissection (PZD) Pipettes are used to cut an opening on the zona of an oocyte mechanically in assisting embryo hatching prior to implantation.

The Zona Drilling Pipettes are used to apply acidic solution onto the zona pellucida of an oocyte, thus creating a hole on the zona and assisting embryo hatching prior to implantation.

The Denuding Pipettes are used to remove cumulus cells attached to the zona prior to micromanipulation procedures and fertilization assessment, and to transfer oocytes and embryos in assisted reproduction procedures.

**Notes:**

2-cell MEA tested: Acceptable at  $\geq 80\%$  blastocysts per embryo.  
Endotoxin tested: LAL acceptable at  $\leq 0.5$  endotoxin units per device.

**Instructions for Use:**

1. Micropipettes should be used by qualified personnel.
2. Micropipettes are for single use only. The used pipettes are contaminated and considered as biohazard materials. They should be disposed properly.
3. Remove a micropipette from its packaging just before use.
4. Check the package for damages before use. Sterility is no longer guaranteed if the pouch is damaged or opened.
5. The pouch can be opened by tearing off one end or by cutting using a pair of scissors.
6. To open the case, hold the case as shown in Figure 1. Make sure the logo engraving is up facing your right hand.
7. Using your left index finger and thumb press the case on both sides marked "IIIII" then using your right index finger and thumb pull the top cover as shown in Figure 2.
8. Hold firmly the pipette in the middle and gently pull it out from the base as shown in Figure 3

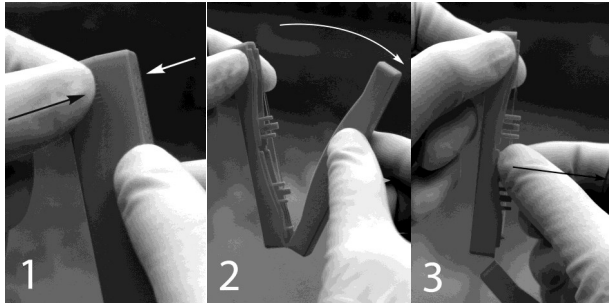
Figure 1

Figure 2

Figure 3

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EC REP

**RMS UK, Ltd**  
28 Trinity Rd. Nailsea, Somerset,  
BS48 4NU, United Kingdom

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**Sunlight Medical, Inc.**
Jacksonville, FL 32257, U.S.A.