

3!FLEX

PRELOADED



3 CLICKS FOR EASY,
SAFE AND PREDICTABLE
INJECTION

MEDI  **NTUR**

Vision of Expertise


MEDJET PL-MA 2.2

Single use injection system for Bi-Flex preloaded monofocal, toric and diffractive progressive hydrophilic IOLs




1st CLICK LOADING

Insert the injector to the wet container

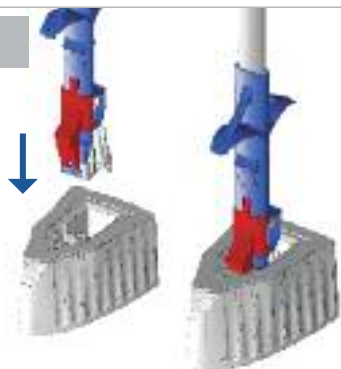


2nd CLICK CLOSING

Fold up the cartridge nozzle

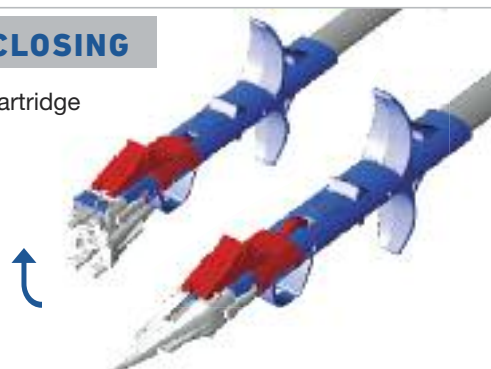


3rd CLICK READY

Push the red stopper forward to the end position and fill visco

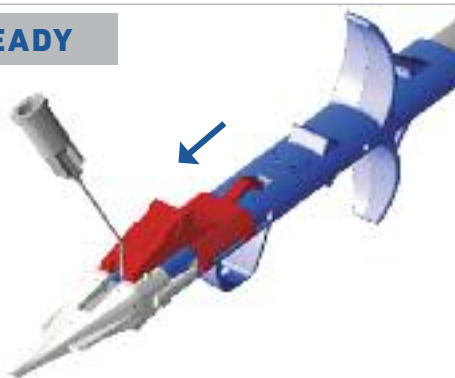

INJECTION

Push the plunger slowly to the end position

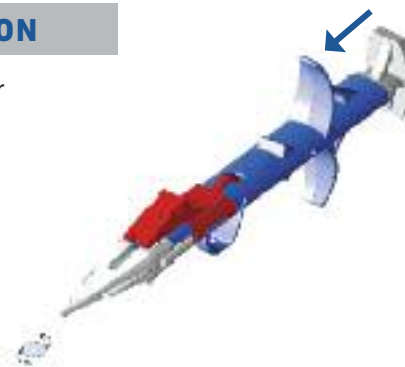

Lens types

Single-piece aspheric IOLs, clear and yellow

- monofocal
- toric
- diffractive progressive

(Patent pending)

Diameters

6.0 mm (optic) / 13.0 mm (overall)

Angulation

0° – asymmetrical design

Material

Copolymer of hydrophobic and hydrophilic monomers

Incision size

2.2 mm

PCO protection

360° Special Square Edge

A-Constant

118.9



677P/677PY

25% water content with UV Absorber + blue light filter (390–470 nm)

Powers available -10.0 D → +9.0 D · (1.0 D steps)
+10.0 D → +30.0 D · (0.5 D steps)



677TP/677TPY

25% water content with UV Absorber + blue light filter (390–470 nm)

Powers available -10.0 D → +30.0 D · (0.5 D steps)
Cylinders available 1.00 D, 1.50 D, 2.25 D, 3.00 D, 3.75 D, 4.50 D



677MPY

25% water content with UV Absorber + blue light filter (390–470 nm)

Powers available 0.0 D → +30.0 D (0.5 D steps)
Addition +3.5 D

LA-IMCMDJPLMA 02 201409