



PAD A CHEEK®

Mask Liner

Quattro Air

**Extra Padding Included,
tuck into the pocket if needed**

- Remove the headgear, and lay the **Mask Liner** out (Photo #1).
- Place the forehead part of the mask into the pockets and extend it through the slits (Photo #2). Pull the Velcro piece off of the mask liner and set it aside for later. We now use a larger forehead pad to cover more of the strap and the connection point than the photos show.
- Pull the side of the mask liner up over the seal and onto the body of the mask (Photo #3).
- Repeat for the other side (Photo #4).
- **Make certain no exhalation vents are covered with the Mask Liner.**
- Smooth out the **Mask Liner** so there are no wrinkles. There is a triangle sewn inside to protect your nose if you need padding for this area. It is easy to trim if you do not want it.
- The mask should look like (Photo #5). Attach the top straps through the forehead pad slits at the top.
- Replace the Velcro over the loop sections to make it fit your mask well. Make sure it does not cover the vents. (Photo #6).

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo 1



Photo 2



Photo 3



Photo 4



Photo 5



Photo 6

The PAD A CHEEK® **Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK® 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com Designed and Sewn in the USA