



PAD A CHEEK[®]

Strap Pad + Aloha



To install the **PAD A CHEEK Strap Pad +** on your mask:

1. On one side, remove the strap from the nasal pillows and remove the top strap from the side strap on that side.
2. From the very top of the pad, slide the strap down the entire length of the pad, matching the shape of the strap.
3. Re-attach the top strap on that side through the slit in the pad.
4. Smooth out the pad along the strap, making sure the seams are on the edges of the strap.
5. Repeat the installation for the other side.

PAD A CHEEK Aloha is made from 100% polyester micro fleece, and can be machine or hand washed with **warm** water. Do not use bleach or softener. **PAD A CHEEK Aloha** is machine (low heat) or air dryable.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: admin@padacheek.com website: www.padacheek.com

