



PAD A CHEEK[®]



Mask Liner Nasal A

Activa LT, Comfort Gel, and Comfort Fusion

**Extra Padding Included,
tuck into the pocket if needed**

- Remove the straps from the mask, and lay the **Mask Liner** out (Photo #1).
- Lift up the sewn strap and put the forehead part of the mask under it (Photo #2).
- Put the forehead pad pockets over the mask forehead support (Photo #3).
- Pull the loose flap hanging down up and over the elbow in the mask, pushing the elbow through the hole (Photo #4).
- Attach the hook portion of Velcro to the loop section of Velcro (Photo #5).
- Reattach the straps to the mask, using the holes in the end of the forehead pad if necessary.
- The last photo shows how the mask liner should be smoothed over the seal to remain leak-free (Photo #6). There is a triangle sewn in to protect your nose, it is easily trimmed smaller.

Make certain no exhalation vents are covered with the Mask Liner.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

#1



#2



#3



#4



#5



#6



The **PAD A CHEEK Mask Liner** is made from 100% polyester (micro fiber and micro fleece), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

website www.padacheek.com

Designed and Sewn in the USA