



PAD A CHEEK[®]

Barrel Cozy Dreamwear



- Place the Barrel Cozy over the nasal pillows.
- Gently pull the pillows through the holes, if the hole seems too small make a small slit in the edge of the hole. (The fabric will not fray)
- Wrap it around the barrel and attach the Velcro to secure it.
- You can place this with the fleece or the micro fiber on the outside.
- **Always make certain the vent of the mask is not covered by the Barrel Cozy.**
- If you have any questions or concerns, please feel free to contact us.

This PAD A CHEEK Barrel Cozy is made of 100% polyester micro fiber and micro fleece.
It is hand or machine washable and dryable (warm water and low heat).

PAD A CHEEK
email admin@padacheek.com

157 Sunset Drive, Stanardsville, VA 22973
website www.padacheek.com

434-985-4003
Designed and Sewn in the USA

