



PAD A CHEEK®

Nose Bridge Protector Style B

US Patent No. 8,235,045 B2



Included are 2 pads with 2 flaps each. Cut the thicker flap off if you want less padding.
Included is a Free extra padding piece.

- Slide the pocket on one end of the [NoseBridge Protector](#) over the end of the mask forehead support.
- Repeat the same procedure for the left side of the forehead piece.
- **NOTE:** If the straps on your mask attach to the forehead pad, you can put the straps through the holes cut in the ends of the [Nose Bridge Protector](#) pad.
- Smooth and straighten the [Nose Bridge Protector](#) until it lies flat and will be comfortable.
- Tuck the flap up inside your mask between the seals to keep it in place. If your mask does not have 2 seals in the cushion to hold the flap into place, you may want to tape it up or trim it shorter.
- If necessary, trim the flap narrower to maintain the mask seal when you wear your mask.

[PAD A CHEEK-Forehead and Nose Protector](#) is made from 100% polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat).

[PAD A CHEEK](#) 434-985-4003
email admin@padacheek.com

157 Sunset Drive, Stanardsville, VA 22973
web site www.padacheek.com Designed and Sewn in the USA

