

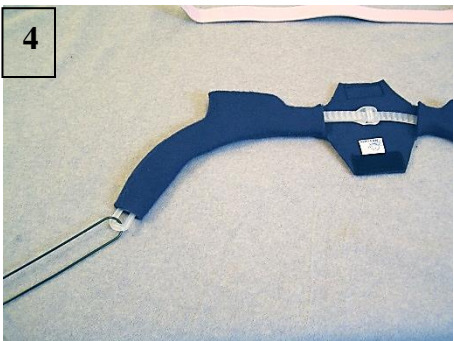
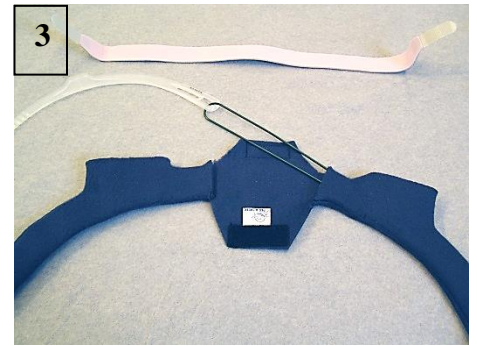


PAD A CHEEK[®]

Strap Pad + Swift FX Over the Top

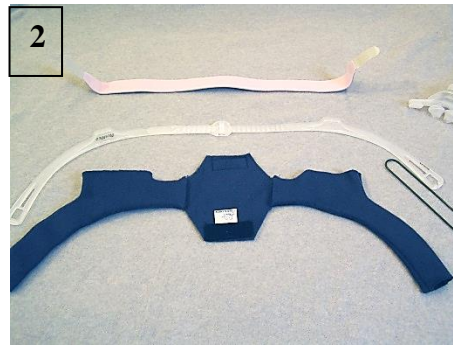

- Lay the **Swift FX Over the Top pad** out in relation to your Swift FX as shown.(Photo #1)
- Remove the straps from the nasal part of the interface on both sides and remove the back strap from the attachment points on the side straps.(Photo #2)
- Put the wire tool through the hole in the strap and push the loops of the wire tool into the **Over the Top pad** starting from the buckle pad. Pull the strap completely through the **Over the Top pad**.(Photos #3)
- Repeat for the other strap of the Swift FX and smooth out the **Over the Top pad**.
- Expose the attachment for the back strap at the each opening and attach the mask's back strap. (Photo #5)
- It should look like this at this point. (Photo #6)
- Wrap the Velcro over the buckle part over the strap. (Photo #7)
- Smooth down the **Over the Top pad** so there are no lumps or wrinkles, (trim the length of the padding if you need to, it will not fray). Re-attach the nasal pillows part and it should look like this.(Photo #8)

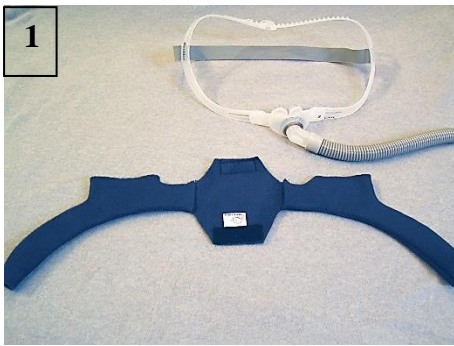

PAD A CHEEK-Swift FX pad is 100% polyester and can be machine or hand washed. If you have questions, please contact us.
PAD A CHEEK LLC 434-985-4003 157 Sunset Drive, Stanardsville, VA 22973
Email: admin@padacheek.com web site: www.padacheek.com Designed and Sewn in the USA