



PAD A CHEEK[®]

Oxygen Cannula Cheek Pad

- These pads may be worn with either the fleece side or the micro fiber side against your skin.
- Put the pad on a table with the desired side face down.
- Lay the cheek section of the cannula tubing along the center of the pad.
- Wrap the sides of the pad around the tubing and secure with the Velcro along the entire length, making certain no Velcro sticks out over the side of the pad.
- Slide the pad up or down the tubing, placing it where it feels the most comfortable against your cheek.

The PAD A CHEEK[®] Oxygen Cannula Cheek Pad is made from 100% polyester (micro fleece and micro fiber), hand wash with warm water and air dry. Wash as often as needed, these pads are very durable.

PAD A CHEEK[®] 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA

