



PAD A CHEEK[®]



Forehead Pad

Style C

FP431/432, Quattro Air,
Mirage FX, Simplus and Profile Lite

Extra Padding Included

- Remove the side strap from the mask forehead support.
- Lay the Forehead Pad on the mask forehead.
- Push the strap through the hole in the end of the Forehead Pad.
- Secure with the Velcro back to the strap going over the outside of the Forehead Pad.
- Repeat the same procedure for the other side.
- Smooth and straighten the Forehead Pad until it lays flat.

PAD A CHEEK-Forehead pad is made from 100% polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat). A second **Forehead Pad** is included for use when washing the first.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: admin@padacheek.com

Web site: www.padacheek.com

