



Opus

# PAD A CHEEK<sup>®</sup>

## Strap Pad Style B – Cotton

Opus, Air Fit F10,  
Air Fit N10, Quattro FX  
And Temple



Temple



Quattro FX



AirFit F10 and N10

### Instructions on Use and Care

#### To install PAD A CHEEK on your Opus mask:

Remove one headgear strap from the side buckle of the Opus. Slide the PAD A CHEEK down the side piece and then re-attach the strap to the side piece of the Opus. Slide the PAD A CHEEK up and smooth it out making sure the seams are away from your face. Repeat for the other side.

#### To Install PAD A CHEEK on your Air Fit F10, Air Fit N10, or Quattro FX mask:

Remove the upper side strap from the plastic arm of the mask. Slide the PAD A CHEEK onto the plastic arm and then reattach the upper side strap to the plastic arm. Slide the PAD A CHEEK up and smooth it out making sure the seams are away from your face. Repeat for the other side.

#### To install PAD A CHEEK on your Temple mask:

Remove the strap from the upper side of your mask. Slide the U shaped end of the wire into the PAD A CHEEK. Put the strap through the wire loop and pull the strap through the PAD A CHEEK fleece tube with the wire. Slide the PAD A CHEEK strap pad far up on the strap. Remove the wire and Re-attach the strap to the mask. Adjust straps for the best fit of the mask. Slide the PAD A CHEEK back down along the strap to the mask, positioning the seam on the side away from the face. Repeat for the other side.

PAD A CHEEK Strap Pad Style B is made from 100% Cotton over micro fleece and can be machine or hand washed in warm water and air or machine dried (low heat).

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: [admin@padacheek.com](mailto:admin@padacheek.com)

Web Site: [www.padacheek.com](http://www.padacheek.com)