

# PAD A CHEEK<sup>®</sup>

## Mask Liner Pico




- Lay out the mask liner (Photo #1).
- Place the wide part of the mask cushion into the bottom of the pocket of the mask. The triangle should be facing up (Photo #2).
- First, pull the flap that has the rough Velcro on it over the hose elbow connection (Photo #3).
- Then, pull the flap that has the soft Velcro on it over the hose elbow connection (Photo #4).
- Overlap the flaps against the mask, attach the Velcro together.
- Make sure the vents are not covered.

If you have any questions, please feel free to call.

**Make certain no exhalation vents are covered with the mask liner.**

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

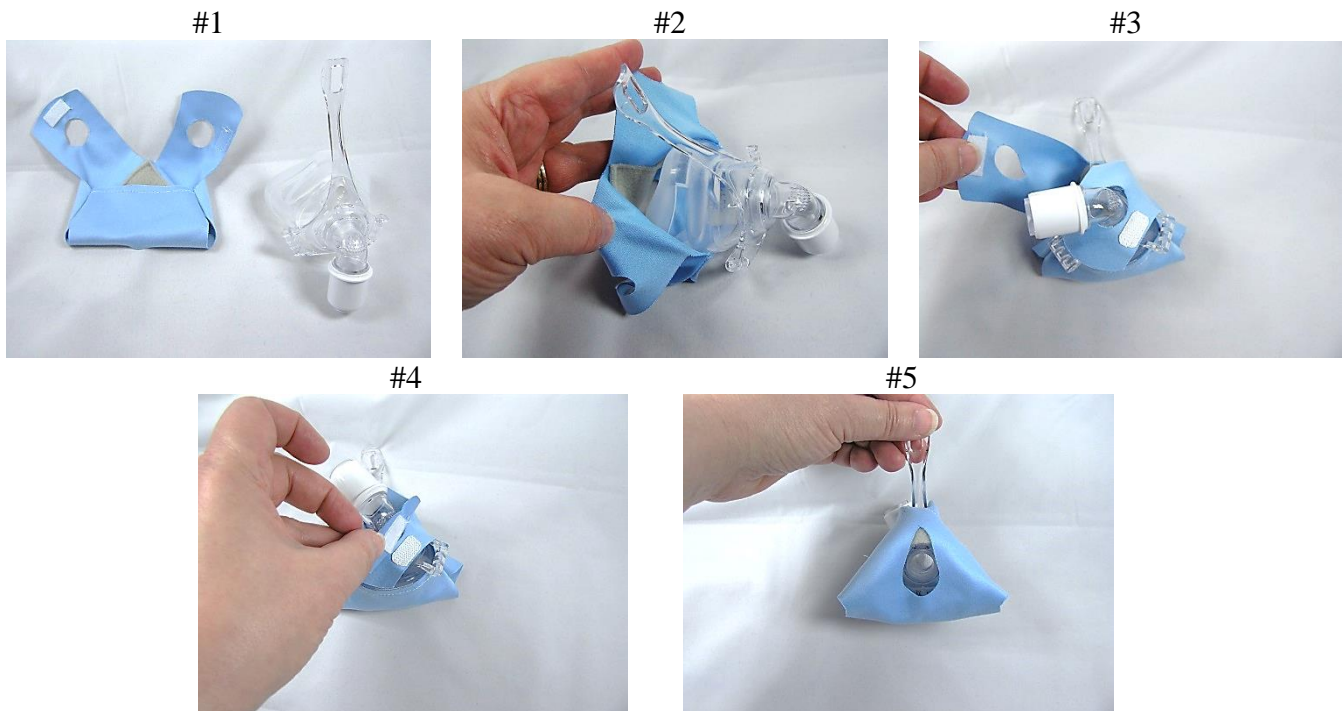

The PAD A CHEEK Pico Mask Liner is made from 100% polyester (micro fiber and micro fleece), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA



# PAD A CHEEK®



## Forehead Pad Pico

(Includes Free additional padding piece. If needed, insert this into the forehead pad before installing on your mask)

- Remove one side strap from the mask forehead support.
- Lay the **Forehead Pad** onto the forehead part of the mask, with the label up and the cushion opening of your mask facing down on the table. (Photo #1)
- Push the strap through the hole in the end of the **Forehead Pad**, then thread the strap through the hole in the top of the mask support (Photo #2) and then out the same side of the **Forehead Pad**
- Secure with the Velcro. Repeat the same procedure for the other side. (Photo #3)
- Smooth the **Forehead Pad** until it lays flat.

**PAD A CHEEK-Forehead pad** is made from 100% polyester micro fleece, hand wash and air dry for best results.



Photo 1



Photo 2



Photo 3

**PAD A CHEEK** 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: [admin@padacheek.com](mailto:admin@padacheek.com)

Web site: [www.padacheek.com](http://www.padacheek.com)