



# PAD A CHEEK<sup>®</sup>



## Strap Pads Original

**Fits lower straps on most Full Face & Nasal Masks**

- Remove one strap from the lower side of your mask and remove the clip.
- (You may use **one, two, or three layers** of padding between your face and the straps, depending on where you slide the wire into the strap pad.)
- Slide the U end of the wire tool all the way through the strap pad and out the other end.
- Put the strap through the closed end of the wire loop and pull strap through the pad.
- Slide the strap pad far up on the strap, remove the wire and re-attach the strap to the mask. Slide the pad down along the strap to the mask.
- Make sure the seam is away from your face when wearing the mask.
- (If necessary, cut strap pad to the best length for you with sharp scissors.)

**PAD A CHEEK<sup>®</sup> Original** is made from 100% polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat or air dry).

**PAD A CHEEK<sup>®</sup>** 434-985-4003 157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA

