

PAD A CHEEK


Strap Pad + Swift LT



Lay the wire tool over the **Swift LT Pad** in this direction. Remove both straps on one side of your Swift LT (from the buckle and the connector in the back).



Overlap the straps by folding one over the other. Put the U end of the wire tool thru the **Swift LT Pad**. Start from the end with no bump.

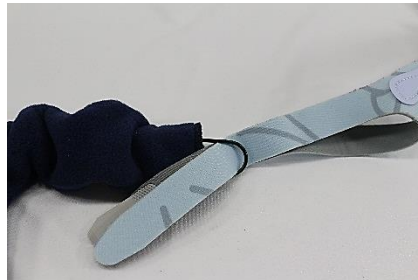

Put the straps **together** thru the wire tool loop and pull both straps completely thru the **Swift LT Pad**.



Put the wire tool through the **back tube**, and put the back strap through the wire tool. Pull the strap and tool thru the side slot in the **Swift LT Pad**. Re-attach the straps to the buckle and the connector. The little extension on your Swift LT goes into the bump.

Repeat for the other side of the Swift Straps. Now put the Swift LT straps back in the buckles as they were before. Smooth the **Swift LT Pad** so it will lay flat.

PAD A CHEEK® Swift LT is made from 100% polyester micro fleece. It can be machine or hand washed with warm water (do not use bleach or softener) and machine (low heat) or air dryable.

PAD A CHEEK® 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

e-mail admin@padacheek.com website www.padacheek.com

Made in the USA