



# PAD A CHEEK<sup>®</sup>



## Strap Pad Set A

**Set contains: one-pair Original Strap Pads  
and one-pair Strap Pad B in Fleece**

- Remove a lower headgear strap from your mask. (if it has a clip, remove the clip and set aside)
- Slide the U end of the wire tool through the longer **Strap Pad Original** and out the end.
- Attach the headgear strap to the wire tool using the Velcro on the strap.
- Pull your headgear strap through the strap pad with the wire tool.
- Remove the wire and re-attach the strap to the mask.
- Smooth the strap pad with the seam away from your face.
- Repeat for the other lower strap on your mask.
- Repeat for the upper straps using the shorter **Strap Pad B** on your mask.
- Adjust straps for best fit. If they are too long, you can trim them to fit with sharp scissors, seams are sewn securely the entire length.

**PAD A CHEEK Strap Pads** are made from 100% polyester micro fleece, and can be washed by hand or machine weekly or more often and air or machine dried.

**PAD A CHEEK** 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email: [admin@padacheek.com](mailto:admin@padacheek.com)

web site: [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA

