

PAD A CHEEK[®]



Mask Liner

Nasal C

Zest, EasyFit

**Extra Padding Included,
tuck into the pocket if needed**

- Remove the headgear and lay the **Nasal Mask Liner** out (Photo #1).
- Put the forehead part of the mask inside the forehead pad (Photo #2/#3).
- Pull the side of the **Nasal Mask Liner** over the body of the mask. (Photo #4)
- Pull the other side of the mask over the body of the mask (Photo #4).
- The pad is to protect the bridge of the nose if you need it.
You can trim it off if you do not need it.
- Smooth the mask liner over the seal so there are no wrinkles (Photo #5).
- Reattach the straps to the upper part of the mask; if necessary, use the holes in the edge of the forehead pad.

Make certain no exhalation vents are covered with the PAD A CHEEK- Nasal Mask Liner.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

#1



#2



#3



#4



#5



The **PAD A CHEEK Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

website www.padacheek.com

Designed and Sewn in the USA