



PAD A CHEEK[®]

Strap Pads Swift FX



- Remove one headgear strap from the side of the nasal pillows assembly.
- Slide the strap pad up the side piece and then re-attach the strap to the side piece of the Swift FX.
- Smooth out the pad, making sure the seams are away from your face.
- Repeat for the other side.

PAD A CHEEK Swift FX Strap Pad is made from 100% polyester micro fleece and can be machine or hand washed in warm water and air or machine dried (low heat).

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com Designed and Sewn in the USA

