



PAD A CHEEK[®]



Strap Pad + Neck

- Lay out the **Neck Pad** so that the seams are face up and the tall tab (with the Velcro loop) is to the top; remove the strap from the lower side of your mask and slide the U-shaped end of the wire tool through the **Neck Pad**.
- Put the strap thru the wire loop, secure with the strap's Velcro end and pull the strap through the **Neck Pad** with the wire.
- Slide the **Neck Pad** as far up on the strap as you can; remove the wire and re-attach the strap to the mask.
- Repeat for the opposite side. Fold the bottom tab (with Velcro hook) up over the strap, then take the tall tab and push it through the hole in the center of the headgear to overlap the lower tab.
- The side tubes and any extra part of the tab of the Neck Pad may be cut to a desired length.

PAD A CHEEK-Neck is made from 100% polyester micro fleece. It can be machine or hand washed with warm water and air or machine (low heat) dried.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA

