

PAD A CHEEK[®]



Strap Pad Pediatric Original Cotton

- Remove the strap from the lower side of your mask.
- Slide the U-shaped end of the wire tool into the **Strap Pad**.
- Put the strap thru the wire loop and pull strap through the **Strap Pad** with the wire tool.
- Slide the **Strap Pad** tube far up on the strap. Remove the wire and re-attach the strap to the mask.
- Adjust straps for best fit.
- Slide the **Strap Pad** back down along the strap to the mask, positioning the seam on the side away from the face.

PAD A CHEEK ORIGINAL is made from cotton over polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat or air dry).

PAD A CHEEK 434-985-4003
157 Sunset Drive, Stanardsville, VA 22973

Email: karen@padacheek.com

web site: www.padacheek.com

Designed and sewn in the USA

