



PAD A CHEEK[®]

Nose Bridge Protector

US Patent No. 8,235,045 B2

Style E



Package contains 1 pad and an extra padding piece

- With the rounded flap hanging down into the mask, slide the pocket on one end of the **Nose Bridge Protector** over the end of the forehead support on the mask.
- Repeat the same procedure for the left side of the forehead piece.
- Smooth and straighten the **Nose Bridge Protector** until it lies flat and will be comfortable against your forehead.
- Tuck the padded end flap up inside your mask between the two layers of the face seal to keep it in place (see photos above). This **Nose Bridge Protector** requires a dual walled cushion on the mask to keep it in place.
- If necessary, trim the flap narrower to maintain the mask seal when you wear your mask.

PAD A CHEEK- Nose Bridge Protector is made from 100% polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat).

PAD A CHEEK 434-985-4003 157 Sunset Drive, Stanardsville, VA 22973



email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA

