



PAD A CHEEK[®]

Strap Pad + DreamWear Strap Pad



- These pads are designed to wrap around the sides of the Dreamwear and secured with the Velcro completely covered.
- Lay out the Strap Pads (Photo #1) (PAD A CHEEK label showing on both pads)
- Place the tab for the back strap in the gap between the Velcro. (Photo #2)
- On the side tube near the hose connection, wrap the sides around the tube and secure with the velcro. (Photo #3)
- Wrap the pad around the lower section and secure with the Velcro.
- Repeat for the other side of the DreamWear.
- The Pad toward your face should be smooth without Velcro. The Velcro should be covered with the fleece and the edges of the pad should butt up together and not overlap.



PAD A CHEEK DreamWear Strap Pad is polyester micro fleece. They can be machine or hand washed weekly or more frequently if necessary in hot soapy water. It is very durable. If you have questions, please contact us.