




PAD A CHEEK[®]

Barrel Cozy Swift LT

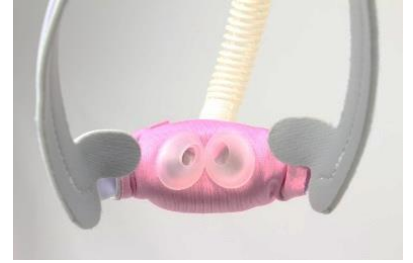

- Place the [Swift LT Barrel Cozy](#) over the Nasal Pillows.
- Gently pull the pillows through the holes, if you need to, cut a slit in the edge of the holes to make it easier to pull the nasal pillows through.
- Secure the Barrel Cozy on the mask with the Velcro and smooth the fabric.

Always make certain the vent on the mask is not covered by the Barrel Cozy

The Pad A Cheek Swift LT Barrel Cozy is made of 100% polyester (micro fiber and micro fleece).

It is hand or machine washable and dryable (warm water and low heat).

If you have any questions or concerns, please feel free to contact me.

PAD A CHEEK
admin@padacheek.com

157 Sunset Drive, Stanardsville, VA 22973 434-985-4003
www.padacheek.com Designed and sewn in the USA

