



# PAD A CHEEK<sup>®</sup>

## Strap Pads GoLife and Swift



### GoLife:

- On one side, remove the strap from the plastic part of the mask.
- Slide the pad over the plastic structure and bunch the pad down near the nasal pillows.
- Re-attach the strap on that side.
- Pull the pad up over the strap and smooth out any wrinkles.
- Make sure the seam is on the side away from your face.
- Repeat the installation for the other side.

**PAD A CHEEK GoLife** is made from 100% polyester micro fleece, and can be hand washed and air dried or machine washed and dried (warm water and low heat).



### For the Swift or Swift II

- On the hose side, gently pull the hose out of the nasal fitting.
- Pull the headgear end off the nasal fitting on that side.
- Remove the rubbery gasket circle and set aside. Slide the round blue end of the strap into the **Strap Pad** and work it all the way through.
- Adjust the pad on the strap until it lies flat and the seam is away from your face.
- Replace the rubbery gasket circle, making sure it is seated completely and looks like the other strap's gasket.
- Re-attach the nasal fitting and replace the hose.
- On the second strap, remove the end plug and follow the same directions for this side, replacing the plug when you have finished.

**PAD A CHEEK Swift Strap Pad** is made from 100% polyester micro fleece, and can be hand washed and air dried or machine washed and dried (warm water and low heat) .

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: [admin@padacheek.com](mailto:admin@padacheek.com) website: [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA