



# PAD A CHEEK<sup>®</sup>

## Mask Liner Hybrid/Liberty



- Pull the nasal pillows of the mask completely through the holes in the **Mask Liner**.
- Place the **Mask Liner** over the seal.
- Remove the lower side straps from your mask.
- Push the lower strap clips through the holes in the **Mask Liner** and clip them back into the mask.
- There is a hole cut for your mouth, you can customize it to your preference.
- **NOTE: If you need to cut the mouth hole larger, you can do this with sharp scissors. Trim a small amount at a time, it is easy to get the hole too large.**

The **PAD A CHEEK Mask Liner** is made from 100% polyester (micro fiber) and can be machine or hand washed with warm water and air or machine dried (low heat). If you have any questions, please contact me.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

Email: [admin@padacheek.com](mailto:admin@padacheek.com)

Web site: [www.padacheek.com](http://www.padacheek.com)

Designed and sewn in the USA

