



PAD A CHEEK[®]

Mask Liner Zest Petite

- Remove the headgear from the mask, and lay the **Mask Liner** out (Photo #1).
- Insert the forehead part of the mask into the pockets of the forehead pad of the mask liner (Photo #2).
- Put the gray straps through the holes in the mask liner (Photo #3), repeat on other side.
- Pull the bottom of the mask liner up over the lower part of the mask (Photo #4).
- Make certain the vent is not covered by the mask liner and smooth out the mask liner on the other side (Photo #5).
- Re-attach the straps to the mask.

Make certain no exhalation vents are covered with the Mask Liner.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo # 1



Photo # 2



Photo # 3



Photo # 4



Photo # 5

The PAD A CHEEK[®] Mask Liner is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK[®] 434-985-4003
157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA