

# PAD A CHEEK<sup>®</sup>



## Strap Pads Pilairo Side Pads

- Unlatch one headgear strap from the side of Pilairo assembly.
- Slide the wire tool up through the **SIDE PAD**.
- Hook the side strap to the tool and pull it through the pad.
- Then latch the strap back on the Pilairo.
- Smooth out the pad, making sure the seams are away from your face.
- Repeat for the other side.

**PAD A CHEEK Pilairo** is made from 100% polyester micro fleece and can be machine or hand washed in warm water and air or machine dried (low heat).

PAD A CHEEK 434-985-4003  
157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com) web site [www.padacheek.com](http://www.padacheek.com) Designed and Sewn in the USA

