

# PAD A CHEEK<sup>®</sup>

## Strap Pad Style B

**Fits Upper Straps on Most Full and Nasal Mask **Except:** AirFit N20 and Wisp**  
**Fits Most Nasal Pillow Strap **Except:** AirFit P10**

This strap pad is versatile and works for many masks.

- In general, for straps that are stiff enough, simply detach the headgear strap of the mask and slide the **Strap Pad** over the strap.
- Reattach the headgear strap. Smooth the pad making sure the seam is on the outside away from your face.
- Repeat for the other side.
- If the straps of your mask are fabric, use the wire tool to aid installation.
- Slide the U-shaped end of the tool through the **Strap Pad** and out the end.
- Attach the headgear strap to the wire tool and pull the headgear strap through the tube.
- Re-attach the headgear strap. Repeat for the other side.

**Strap Pad Style B** is made from micro fleece. For best results hand wash and air dry weekly or more often. These pads will last over 6 months.

PAD A CHEEK 434-985-4003 157 Sunset Drive, Stanardsville, VA 22973  
Email: [admin@padacheek.com](mailto:admin@padacheek.com) [www.padacheek.com](http://www.padacheek.com) Designed and Sewn in the USA

