



# PAD A CHEEK<sup>®</sup>

## Mask Liner Wisp



- Remove the seal from the mask
- Place the seal into the mask liner (Photo #2 & #3). The wide part of the mask cushion should be against the label.
- Pull the flap over the hole of the seal, pulling it over the edge of the flange. It will stretch.
- Repeat this for the other flap (Photo #5).
- Snap the headgear over the hole and the flaps of the mask liner (Photo #6).

**Make certain no exhalation vents are covered** with the **PAD A CHEEK- Wisp Mask Liner**.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

Photo #1



Photo #2



Photo #3



Photo #4



Photo #5



Photo #6



The **PAD A CHEEK Wisp Mask Liner** is made from 100% polyester microfiber, and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK 434-985-4003  
157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA