



# PAD A CHEEK<sup>®</sup>

## Mask Liner Amara View



- Remove the top and bottom straps from the mask.
- Lay the liner out (Photo #1).
- Place the top of the mask into the liner (Photo #2).
- Pull the sides of the mask liner over the places where the top straps attach (Photo #3).
- The side where the hose attaches should now look the same as (Photo #4).
- Pull the legs up and secure with the Velcro (Photo #5).
- **Make certain no exhalation vents are covered with the Mask Liner.**
- Smooth the **Mask Liner** over the seal.
- The mask should look like photo #6. Attach the straps to your mask. You can attach the straps before putting the “legs” up if you wish.

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo 1



Photo 2



Photo 3



Photo 4



Photo 5



Photo 6

The **PAD A CHEEK<sup>®</sup> Mask Liner** is made from 100% polyester micro fiber, and can be machine or hand washed with soap and warm water. If you have any questions, please contact me.

**PAD A CHEEK<sup>®</sup>** 434-985-4003  
157 Sunset Drive, Stanardsville, VA 22973