



PAD A CHEEK[®]

Black Out Curtain DreamStation by Philips



- Remove the hose from the top of your machine.
- Place the hole of the Black Out Curtain over the hose connection on the machine.
- Replace the hose on the CPAP machine.
- Make certain the hole in the Black Out Curtain does not interfere with the connection if you have a heated hose.
- Drape the Black Out Curtain over the front display of the CPAP machine.
- It can be moved aside to view your data without falling off.

The **PAD A CHEEK Black Out Curtain** is made from 100% polyester micro fiber fabric and can be machine or hand washed and machine or air dried (warm water, low heat).

PAD A CHEEK
Email: admin@padacheek.com

434-985-4003 157 Sunset Drive, Stanardsville, VA 22973
website: www.padacheek.com

Made in the USA