



PAD A CHEEK[®]

Strap Pads Original Cotton



- Remove the strap from the lower side of your mask. Slide the U shaped end of the wire through the **Strap Pad**.
- Put the strap thru the wire loop, and secure with the Velcro.
- Pull strap through the **Strap Pad** fleece tube with the wire.
- Slide the **Strap Pad** tube far up on the strap.
- Remove the wire and re-attach the strap to the mask.
- Adjust straps for best fit.
- Slide the **Strap Pad** back down along the strap to the mask, positioning the seam on the side away from the face.
- Smooth out any lumps on the side toward your face

PAD A CHEEK is made from 100% cotton exterior and polyester micro fleece interior and can be machine or hand washed and dried (warm water, low heat or air dry).

434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Made in the USA

