



PAD A CHEEK®



Strap Pad + Strap Pad Y Wisp and N20 straps

- On one side, release the headgear and lay the pad (Photo #1) with the label showing.
- Slide the long part of the headgear into the upper tube, slide it up and put the lower part of the headgear into the lower tube (Photo #2).
- Fold the Velcro parts over the headgear to secure (without overlapping) (Photo #3).
- Slide the pad down on the upper long arm to expose the slot for the headgear and attach the headgear (Photo #5).
- Slide the lower headgear into the hole of the lower tube (for the N20 the magnet will attach) (for the Wisp, slide the tube up out of the way and attach the strap).
- Smooth the pad over the straps so there are no wrinkles on the face side. (Photos #6 & #7).
- Repeat for the other side of the headgear.

These photos are of the AirFit N20, the headgear on the Wisp is similar and installation is the same.



Photo #1



Photo #2



Photo #3



Photo #4



Photo #5



Photo #6



Photo #7

The PAD A CHEEK® Strap Pad Y is made from polyester micro fleece.

Hand wash weekly, or more often, with any soap you would use on your face and air dry for best results.

email admin@padacheek.com

PAD A CHEEK® 434-985-4003

web site

157 Sunset Drive, Stanardsville, VA 22973

www.padacheek.com

Designed and Sewn in the USA