



# PAD A CHEEK<sup>®</sup>

## Mask Liner

### FFA general instructions

**Extra Padding Included**

**Tuck into the pocket if needed**

- Remove the straps and hose from the body of the mask, and lay the **FFA-Medium Mask Liner** out (Photo #1)
- Lift up the sewn strap; insert the forehead part of the mask under it (Photo #2)
- Put the forehead part into the pad at the top (Photo #3)
- Pull the flap up over the hose connection and attach to the Velcro (Photo #4). Adjust the size by turning the Velcro tab over and connecting the reverse side to make it smaller.
- Smooth out the **Mask Liner**. If you need the hole larger, you can trim the hole by cutting little slits in the bottom and extending these slits, then cut the fabric away to allow your mouth to be uncovered (Photo #5).

**Do this carefully; it is very easy to get the hole too large.**

- Re-attach the straps to the mask.

**Make certain no exhalation vents are covered with the Mask Liner**

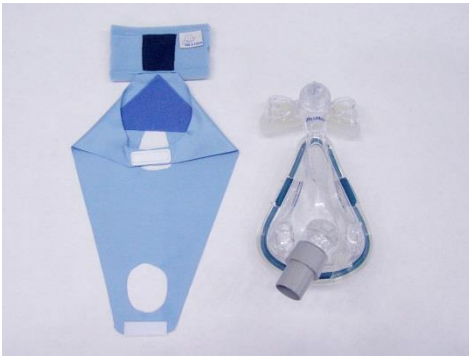


Photo # 1



Photo # 2

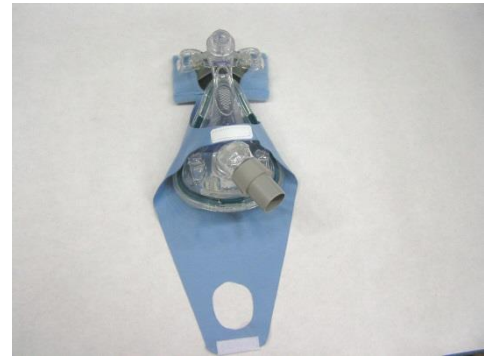


Photo # 3



Photo # 4



Photo # 5

The **PAD A CHEEK<sup>®</sup> Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water as often as needed. If you have any questions, please contact us.

PAD A CHEEK<sup>®</sup> 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and Sewn in the USA