



PAD A CHEEK[®]

Mask Liner FFD

AirFit F10, AirFit F20, AirTouch F20

**Extra Padding Included,
tuck into the pocket if needed**



- Remove the top and bottom straps from the mask, leave the cushion on the mask body.
- Place the arms of the mask through the slits (photo #2) with the Velcro showing on the hose connection side of the mask. The hole for your nose/mouth should be over the cushion of the mask.
- Pull the top curved seam of the mask liner up over the seal at the top of the body of the mask.
- Smooth the sides down on both sides (Photo #3), note how the attachment point is exposed in the hole.
- Pull the legs up and secure with the Velcro (Photos #4 and #5).
- Smooth the **Mask Liner** over the seal and make sure the pad inside (to protect your nose bridge) is smooth. The mask should look like this (Photo #6). Attach the straps to your mask.

Make certain no exhalation vents are covered with the Mask Liner

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. It is very easy to get the hole too large.

These photos are with the AirFit F20 mask, but it fits both the F10 and the F20



Photo 1



Photo 2



Photo 3



Photo 4

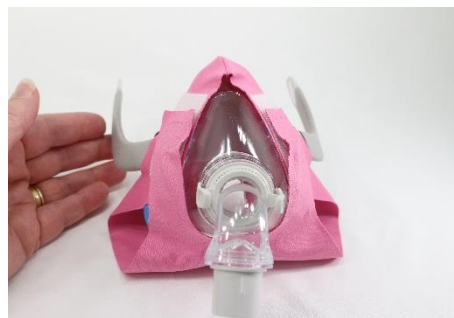


Photo 5



Photo 6

The **PAD A CHEEK[®] Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK[®] 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973