



PAD A CHEEK[®]

Nose Bridge Protector Style C

US Patent No. 8,235,045 B2

Included is a Free extra padding piece.



- Remove the side strap from the mask forehead support.
- Lay the **Nose Bridge Protector** on the mask forehead pad with the flap hanging down into the mask.
- Push the strap through the hole in the end of the **Nose Bridge Protector**.
- Secure with the Velcro back to the strap going over the outside of the Nose Bridge Protector.
- Repeat the same procedure for the other side of the headgear.
- Smooth and straighten the **Nose Bridge Protector** until it lays flat.
- Tuck the Nose Bridge Protector flap into the seal to hold it in place. If your mask does not have a dual walled cushion, you may need to tape it up inside the mask or trim it shorter.
- The flap may be trimmed as needed to rest on the bridge of your nose when you wear the mask.

PAD A CHEEK-Nose Bridge Protector is made from 100% polyester micro fleece and can be machine or hand washed and dried (warm water, low heat).

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA

