



PAD A CHEEK[®]

Mask Liner

FFB

Quattro FX

**Extra Padding Included,
tuck into the pocket if needed**

- Remove the headgear from the mask, and lay the **Mask Liner** flat (photo #1).
- Squeeze the side pieces together and slide under the sewn strap (photo #2). This feels awkward.
- Then pull the sewn strap with the Velcro down onto the body of the mask (photo #3) This is a tight fit.
- Pull the top of the mask liner over the top of the mask line (photo #4).
- Put the elbow of the mask thru the round hole in the bottom of the liner.
- Attach the Velcro. If the mask liner needs to be smaller, roll the bottom Velcro over before attaching it (Photo #5). **Make certain no exhalation vents are covered with the Mask Liner.**
- Smooth the **Mask Liner** over the cushion and under the “arms” of the mask. (photo #6)
- Attach the straps to your mask.

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo 1



Photo 2



Photo 3



Photo 4



Photo 5



Photo 6

The **PAD A CHEEK[®] Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. We recommend washing it once a week (more frequently if you have oily skin). If you have any questions, please contact us.

PAD A CHEEK[®] 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com Designed and Sewn in the USA