

# PAD A CHEEK<sup>®</sup>



## Strap Pads Pediatric Original

- Remove the strap from the lower side of your mask.
- Slide the U-shaped end of the wire tool into the **Strap Pad**.
- Put the strap thru the wire loop, Pull strap through the **Strap Pad** with the wire tool.
- Slide the **Strap Pad** tube far up on the strap.
- Remove the wire and re-attach the strap to the mask.
- Adjust straps for best fit.
- Slide the **Strap Pad** back down along the strap to the mask, positioning the seam on the side away from the face.

**PAD A CHEEK** is made from 100% polyester micro fleece, and can be machine or hand washed and dried (warm water, low heat or air dry).

**PAD A CHEEK** 434-985-4003  
157 Sunset Drive, Stanardsville, VA 22973

Email: [karen@padacheek.com](mailto:karen@padacheek.com)

web site: [www.padacheek.com](http://www.padacheek.com)

Designed and sewn in the USA

