



# PAD A CHEEK<sup>®</sup>



## Forehead or Buckle Pad

### Style B

### Fits MOST MASKS

This package contains 2 Forehead Pads and one free additional padding piece that can slip inside for more padding.



## FOREHEAD PAD

- Slide the pocket on one end of the **Forehead Pad** over the end of the forehead support on the mask.
- Repeat the same procedure for the other side of the forehead piece.

NOTE: If the straps on your mask attach to the forehead piece, you can put the straps through the holes cut in the end of the Forehead Pad. Smooth and straighten the Forehead Pad until it lies flat and is comfortable against your forehead.



## BUCKLE



- Remove the strap from one side of the buckle.
- Insert the strap thru the hole in the edge of the Buckle Pad.  
(Material will stretch or you may want to enlarge the hole slightly.)
- Re-attach the strap to the buckle.
- Repeat these steps for the other buckle strap.
- Smooth the **Buckle Pad** so it lies flat.
- You can leave the strap ends out so you can adjust them if necessary without removing the Buckle Pad.

This **PAD A CHEEK Buckle Pad** is made from 100% polyester micro fleece, and can be machine or hand washed (warm water and No bleach); or machine or air dryable (low heat).

**PAD A CHEEK** 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and sewn in the USA