



PAD A CHEEK[®]

Mask Liner Nasal D

Eson, Eson 2, and Mirage FX

**Extra Padding Included,
tuck into the pocket if needed**

- Remove the headgear, and lay the **Mask Liner** out (Photo #1).
- Lift the sewn strap, insert the forehead part of the mask under it (Photo #2).
- Then put the forehead part up to the pad at the top (Photo #3).
- Pull the “legs” over the mask and secure with the Velcro (Photo #4).
- At the forehead support, slide the headgear strap through the holes in the sides and attach to the mask. (Photo # 5)
- Make certain the vent is not covered by the mask liner and smooth out the mask liner on the other side (Photo #6).
- There is a triangle sewn in to protect your nose, it is easy to trim if needed.

Make certain no exhalation vents are covered with the Mask Liner.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo # 1



Photo # 2



Photo # 3



Photo # 4



Photo # 5



Photo # 6

The **PAD A CHEEK[®] Mask Liner** is made from 100% polyester (micro fleece and micro fiber) and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK[®] 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

www.padacheek.com

Designed and Sewn in the USA